



INDICATIONS FOR USE

Acuicyn™ Antimicrobial Eyelid & Eyelash Hygiene solution is intended to be used as a cleanser in the management of skin abrasions, lacerations, minor irritations, cuts and intact skin, and for debridement of wounds such as stage I-IV pressure ulcers, partial and full thickness wounds, diabetic foot ulcers, post-surgical wounds, first- and second-degree burns, grafted and donor sites.

DIRECTIONS FOR USE

Use Acuicyn™ Antimicrobial Eyelid & Eyelash Hygiene solution to clean lids and lashes to remove encrustation and debris from lid margin.

1. Remove any make up or lotions around your eyes.
2. Apply Acuicyn™ Antimicrobial Eyelid & Eyelash Hygiene solution to a 100% cotton pad until moistened. If using a cotton swab, apply 1 spray of Acuicyn™ Antimicrobial Eyelid & Eyelash Hygiene solution to the tip of the swab.
3. Close your eyes.
4. Using a horizontal motion, gently wipe the base of all the upper lid lashes at least three times.
5. With eyes open, look up and wipe the base of the lower lashes with a horizontal motion at least three times.
6. Gently clean your lower lid from your nose to the side of your face.
7. With a new cotton round pad or swab, repeat the other eye.
8. Use twice daily or as needed/recommended.

USE PROCEDURES

Instructions for Use of Spray Pump:

1. Remove original cap from bottle.
2. Inset pump sprayer into Acuicyn™ Antimicrobial Eyelid & Eyelash Hygiene solution bottle and screw sprayer down securely.
3. Hold upright to spray Acuicyn™ Hygiene Eyelid & Eyelash Hygiene solution onto cotton pads.

CONTACT TIME

Apply Acuicyn™ Antimicrobial Eyelid & Eyelash Hygiene solution as directed by your healthcare practitioner.

HANDLING

Non-toxic, non-sensitizing and non-irritating to the skin and eyes. No special handling precautions required.

INGREDIENTS

0.01% pure hypochlorous acid in dilute saline

STORAGE

Caution: DO NOT FREEZE. Non-flammable. Store with pump closed tightly after each use in its original container at room temperature away from direct sunlight and heat.

Caution: Federal law restricts this device to sale by or on the order of a physician.

TIME KILL TESTING

Name of Organism	Log ¹⁰ Reduction	Time to Kill	Percent Reduction
<i>Acinetobacter baumannii</i>	6.3692	30 seconds	99.9999%
<i>Bacteroides fragilis</i>	7.6435	30 seconds	99.9999%
<i>Candida albicans</i>	6.3345	30 seconds	99.9999%
<i>Clostridium difficile</i> (spores)	5.8779	1 minute	99.9999%
<i>Enterobacter aerogenes</i>	6.0881	30 seconds	99.9999%
<i>Enterococcus faecalis</i> -VRE	6.3646	30 seconds	99.9999%
<i>Enterococcus faecium</i> -VRE MDR	6.5119	30 seconds	99.9999%
<i>Escherichia coli</i>	5.6990	30 seconds	99.9998%
<i>Escherichia coli</i> ESBL (Carbapenemase +)	5.9865	30 seconds	99.9999%
<i>Haemophilus influenzae</i>	5.1775	30 seconds	99.9993%
<i>Klebsiella oxytoca</i> -MDR	6.0492	30 seconds	99.9999%
<i>Klebsiella pneumoniae</i>	6.1430	30 seconds	99.9999%
<i>Klebsiella pneumoniae</i> KPC3 (Carbapenemase +)	6.0952	30 seconds	99.9999%
<i>Micrococcus luteus</i>	5.8420	30 seconds	99.9999%
<i>Proteus mirabilis</i>	6.2028	30 seconds	99.9999%
<i>Pseudomonas aeruginosa</i>	5.8096	30 seconds	99.9998%
<i>Serratia marcescens</i>	5.9978	30 seconds	99.9999%
<i>Staphylococcus aureus</i> -MRSA	6.3454	30 seconds	99.9999%
<i>Staphylococcus aureus</i>	6.2266	30 seconds	99.9999%
<i>Staphylococcus epidermidis</i>	6.0233	30 seconds	99.9999%
<i>Staphylococcus haemolyticus</i>	5.9112	30 seconds	99.9999%
<i>Staphylococcus hominis</i>	5.4456	30 seconds	99.9996%
<i>Staphylococcus saprophyticus</i>	5.9590	30 seconds	99.9999%
<i>Streptococcus pyogenes</i>	6.7160	30 seconds	99.9999%

Note: The solution contains hypochlorous acid as preservatives to reduce/prevent the growth of microorganisms within the solution.

HOW SUPPLIED

40 mL and 80 mL with spray pump
40 mL commercial product code: 868489000276
80 mL commercial product code: 868489000283

DISPOSAL

Non-toxic and safe to the environment. No special disposal considerations required. Container may be recycled.

WARNING

For external use only. Not for injection.

Sonoma Pharmaceuticals, Inc.

Petaluma, CA 94954 USA
Phone: 1-800-759-9305
www.sonomapharma.com
www.acuicyn.com